

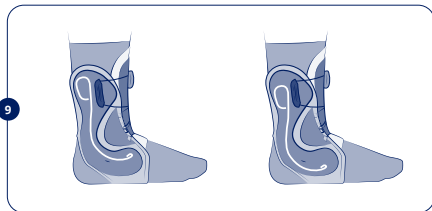
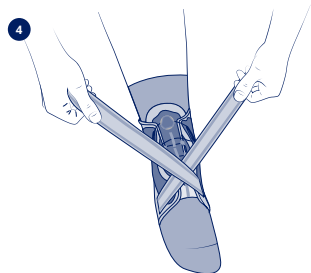
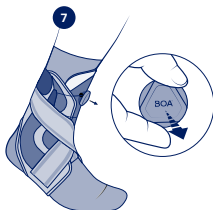
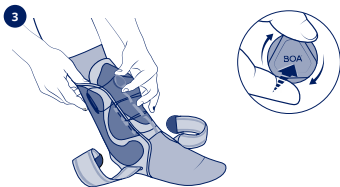
THUASNE**SPORT**

fr	Chevillère stabilisatrice Thuasne Sport BOA	6
en	Thuasne Sport BOA stabilising ankle brace	8
de	Stabilisierende Knöchelbandage Thuasne Sport BOA	9
nl	Thuasne Sport BOA stabiliserende enkelbrace	12
it	Cavigliera stabilizzatrice Thuasne Sport BOA	14
es	Tobillera estabilizadora Thuasne Sport BOA	16
pt	Tornoeleira estabilizadora Thuasne Sport BOA	18
da	Stabiliserende ankelbind Thuasne Sport BOA	20
fi	Vakauttava Thuasne Sport BOA-nilkkatuki	22
sv	Thuasne Sport BOA ankelortos för stöd	23
el	Σταθεροποιητική επιστραγαλίδα Thuasne Sport BOA	25
cs	Stabilizační kotníková ortéza Thuasne Sport BOA	27
pl	Stabilizująca orteza skokowo-stopowa Thuasne Sport BOA	29
lv	Thuasne Sport BOA stabilizējošā potītes ortoze	31
lt	Stabilizuojantis kulkšnies įtvartis „Thuasne Sport BOA“	33
et	Thuasne Sport BOA stabiliseeriv hüppeliigese tugi ...	35
sl	Stabilizacijska opornica za gleženj Thuasne Sport BOA	36
sk	Stabilizačná ortéza členku Thuasne Sport BOA	38
hu	Thuasne Sport BOA stabilizáló bokarögzítő	40
bg	Стабилизираща ортеза за глезен Thuasne Sport BOA	42
ro	Glezniere stabilizatoare Thuasne Sport BOA	44
ru	Стабилизирующий голеностопный ортез Thuasne Sport BOA	46
hr	Steznik za stabilizaciju gležnja Thuasne Sport BOA ...	48
zh	Thuasne Sport BOA 稳定用护踝	50
ar	Thuasne Sport BOA دعامة تثبيت الكاحل	53

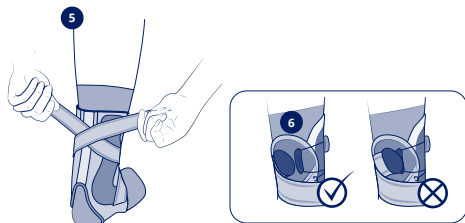BOA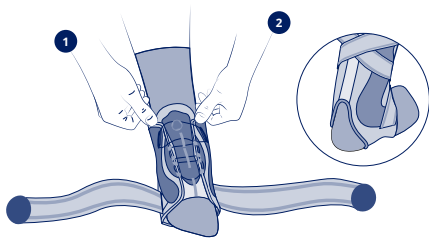

S

19 - 22

M

22 - 26

L

26 - 30

fr	Appuyez pour enclencher	Tournez pour serrer	Tirez pour desserrer rapidement
en	Push in to engage	Turn to tighten	Pull up for quick release
de	Zum Einrasten drücken	Zum Festziehen drehen	Zum Öffnen ziehen
nl	Indrukken om aan te draaien	Draaien om aan te spannen	Optrekken om snel los te maken
it	Spingere per fissare	Girare per stringere	Tirare per rilasciare
es	Empuje para activar	Gire para apretar	Tire para liberar rápidamente
pt	Premir para acionar	Rodar para apertar	Puxar para cima para soltar rapidamente
da	Skub ind for at fastgøre	Drej for at stramme	Træk op for hurtigt at løse
fi	Kytke painamalla	Kiristä kiertämällä	Vapauta nopeasti vetämällä
sv	Tryck in för att sätta fast	Vrid för att dra åt	Dra uppåt till för att lossa snabbt
el	Πιέστε προς τα μέσα για ενεργοποίηση	Στρέψτε για σύσφιξη	Τραβήξτε προς τα έξω για γρήγορη απελευθέρωση
cs	Stisknutím zavčakněte	Otočením utáhněte	Zvednutím povolte
pl	Wcisnąć, aby wprowadzić	Przekręcić, aby zacisnąć	Pociągnąć w górę, aby szybko zwolnić
lv	Nospiediet, lai nofiksētu	Pagrieziet, lai savilktu	Pavelciet, lai ātri atbrīvotu
lt	Paspauskite, kad sujungtumėte	Pasukite, kad suverztumėte	Patraukite, kad greitai atlaisvintumėte
et	Sulgemiseks vajutage	Pingutamiseks keerake	Kiireks lahti tegemiseks tõmmake
sl	Pritisnite, da vklopite	Vrtite, da zategnete	Povlecite gor, da se hitro odklopi
sk	Zatlačením zaistite	Otočením utiahnite	Potiahnutím rýchlo uvoľnite
hu	Rögzítéshez nyomja meg	Szorításához fordítsa el	Gyors kioldáshoz húzza ki
bg	Натиснете, за да поставите	Завъртете, за да затегнете	Издърпайте нагоре за бързо освобождаване
ro	Apăsati pentru a cupla	Rotiți pentru a strânge	Trageți în afară pentru a elibera rapid
ru	Нажмите для установки в рабочее состояние	Поверните, чтобы затянуть	Потяните, чтобы расстегнуть
hr	Pritisnite za zaključavanje	Okrnite za zatezanje	Povucite za brzo otpuštanje
zh	向下压启用	旋转拧紧	向上拔起快速释放
ar	اضغط للتريب	أدر للتشد	اسحب لفتح بسرعة

fr CHEVILLÈRE STABILISATRICE THUASNE SPORT BOA

Description/Destination

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THUASNE SPORT BOA STABILISING ANKLE BRACE

Description/purpose

This device is an ankle brace providing joint stabilisation. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - Polyester - Neoprene - Elastane - Polyurethane - Stainless steel.

Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid frame and elasticated webs to give stability and immobilisation to the ankle.

Fast, durable and micro-adjustable Boa Fit System.

A slim product that will fit easily inside a shoe.

This is a bilateral product.

Indications

Ankle immobilisation.

Moderate to severe ankle sprain.

Joint laxity/chronic instability prevention during sport activities resumption.

Tendinopathy treatment.

Conservative treatment of Achilles tendon injuries.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. For hygiene, security and performance reasons, do not re-use the product for another patient.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral straps. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious event relating to the device must be reported to the manufacturer and competent authority in the Member State in which the user and/or the patient is based.

Instructions for use/Application

It is recommended that the product be worn in a closed sports shoe when practicing sport, and at night for protection in order to limit and avoid sudden movements.

Before putting the ankle brace on, make sure the laces are completely unfastened.

- Fitting the ankle brace:** Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose. ① ②
- Fastening the ankle brace:** Tighten the ankle brace using the Boa Fit System closure system; to do this, push the wheel in until you hear a 'click', then turn clockwise until the correct level of support is obtained. ③
- Fitting the straps:** Take a strap in each hand, then pass them over each other across the upper foot ④. Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric ⑤. Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion. ⑥
- Releasing the ankle brace:** Pull on the wheel until you hear the 'click' to unfasten the laces. ⑦
- Fitting inside a shoe:** Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe's closure mechanism. ⑧
In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace. ⑨

Care/maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Make sure to remove the lateral stays before washing. Replace in the original location before next use. Close the self-fastening tabs and close the product (Boa Fit System) before washing. If possible, use a washing net. Dry in a ventilated area, away from sources of heat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033501 (2022-02)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186