THUASNE

Epi-med[®] (35087)

- de **Gebrauchsanweisung - Epicondylitis-Spange**
- en **Instructions for use - Epicondylitis clasp**
- fr **Consignes d'utilisation - Bracelet anti-épicondylite**
- it **Istruzioni per l'uso - Pressore per epicondilitite**
- es **Instrucciones de utilización - Pulsera para epicondilitis**
- pt **Instruções de uso - Braçadeira para epicondilitite**
- nl **Gebruiksaanwijzing - Epicondylitis brace**
- da **Brugsanvisning - Epicondylitis-bandage**
- sv **Bruksanvisning - Epikondylitarmbågsstöd**
- fi **Käyttöohje - Tenniskyynärpäätukki**
- el **Οδηγίες χρήσης - Περιαγκώνιο επικονδυλίτιδος**
- cs **Návod k použití - Epikondylární páska**
- pl **Sposób użytkowania - Orteza epikondylopatii**
- et **Kasutusjuhend - Põndapealise põletiku pannaal**
- lv **Lietošanas instrukcija - Epikondilīta ortoze**
- lt **Naudojimo instrukcija - Įtvaras nuo epikondilito**
- sk **Návod na použitie - Epikondylická páska**
- sl **Navodila za uporabo - Epikondilitis zaponka**
- hu **Használati utasítás - Epicondylitis ortézis**
- ro **Instrucțiuni de utilizare - Orteză epicondilită**
- bg **Инструкция за употреба - Епикондиларна лента**
- hr **Upute za uporabu - Steznik za teniski lakat**

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Epicondylitis clasp**Purpose:**

Epi-med® is an Epicondylitis clasp with silicone pads. The orthosis may only be used for elbow treatment. This product requires qualified and individual adaptation to the patient by specialist staff.

How it works:

The **Epi-med®** epicondylitis clasp compresses the forearm musculature below the elbow, thus bringing mechanical relief to the tendon attachments at the epicondyle (bone projection) and relieving the extensor and flexor musculature. This biomechanical adjustment of the areas under strain in the elbow leads to a noticeable improvement, frequently to complete relief from pain.

Indications:

- Tendinopathy (inflammation of the tendon attachments) in the elbow
- Medial epicondylitis (golfer's elbow)
- Lateral epicondylitis (tennis elbow)

Contraindications:

Large skin lesions or diseases, extended hematomas or severe swelling

Side effects:

No side effects are as yet known for correct use.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the

Member State in which the user and/or patient are/is domiciled.

Directions for applying:

Position the opened clasp from the outside onto the outer forearm so that the larger of the two pads rests on the upper side of the forearm and is turned slightly towards the body. The **Epi-med**[®] clasp is then closed using the elastic strap. The strap is fastened to the hook and loop fastener firmly but not tightly. Tighten the strap up to the tactile mark **(A)** engraved on the fleece belt strap. The **Epi-med**[®] clasp should not exert tangible pressure when the musculature is relaxed, compression should be felt only when the forearm musculature is tensed.

Use of the silicone compression pads (flexipads):

If the clasp pressure is not sufficient, it is possible to increase it by using the enclosed flexipads. To do this, remove the fastened padding partially in the corresponding area and position the flexipad in the area which is bordered by the Velcro fastening **(B)**. Then re-fasten the padding and press on firmly.

Care:

To clean the padding, remove it and wash it with a mild detergent at 30°C. Carefully squeeze the water out and dry the padding away from a heat source. The plastic clasp and the belt can be cleaned with a damp cloth.

Material composition:

Plastic clasp	Polymer plastic
Velcro fastenings	100% nylon
Padding	Textile 100% polyamide Open-cell PU foam
Pressure pads	100% silicone
Elastic belt	90% polyamide, 10% elasthane

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd**
THUASNE DEUTSCHLAND GmbH