**THUASNE**

# Malleo Dynastab®

<b>fr</b>	<b>Attelle stabilisatrice de cheville</b>	<b>5</b>
<b>en</b>	<b>Stabilising ankle brace</b>	<b>6</b>
<b>de</b>	<b>Orthese zur Stabilisierung des Sprunggelenks</b>	<b>8</b>
<b>nl</b>	<b>Stabilisatiebrace voor de enkel</b>	<b>9</b>
<b>it</b>	<b>Tutore stabilizzatore per caviglia</b>	<b>11</b>
<b>es</b>	<b>Férula estabilizadora de tobillo</b>	<b>13</b>
<b>pt</b>	<b>Tala de estabilizadora do tornozelo</b>	<b>14</b>
<b>da</b>	<b>Stabiliserende ankelskinne</b>	<b>16</b>
<b>fi</b>	<b>Stabiloiva nilkkatuki</b>	<b>17</b>
<b>sv</b>	<b>Stabiliseringsskena för ankeln</b>	<b>19</b>
<b>el</b>	<b>Νάρθηκας σταθεροποίησης ποδοκνημικής</b>	<b>20</b>
<b>cs</b>	<b>Stabilizační ortéza kotníku</b>	<b>22</b>
<b>pl</b>	<b>Orteza do stabilizacji kostki</b>	<b>23</b>
<b>lv</b>	<b>Potītes stabilizēšanas saite</b>	<b>25</b>
<b>lt</b>	<b>Stabilizuojantis kulkšnies įtvaras</b>	<b>27</b>
<b>et</b>	<b>Hüppeliigese fikseerimise lahas</b>	<b>28</b>
<b>sl</b>	<b>Opornica za stabilizacijo gležnja</b>	<b>30</b>
<b>sk</b>	<b>Stabilizačná členková dlaha</b>	<b>31</b>
<b>hu</b>	<b>Stabilizáló bokarögzítő</b>	<b>32</b>
<b>bg</b>	<b>Стабилизираща шина за глезен</b>	<b>34</b>
<b>ro</b>	<b>Atelă stabilizatoare pentru gleznă</b>	<b>36</b>
<b>ru</b>	<b>Стабилизирующий ортез для лодыжки</b>	<b>37</b>
<b>hr</b>	<b>Udlaga za stabilizaciju gležnja</b>	<b>39</b>
<b>zh</b>	<b>踝部穩定夾板</b>	<b>40</b>
<b>ar</b>	<b>جبيرة تثبيت الكاحل</b>	<b>43</b>

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

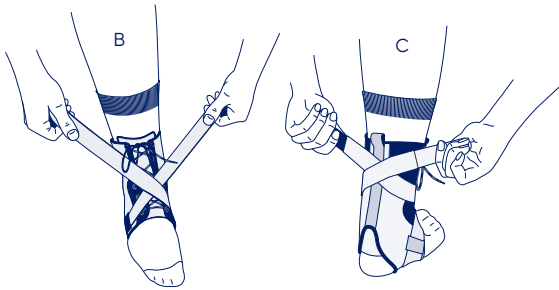
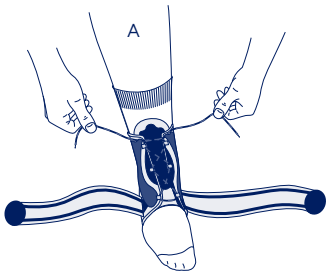
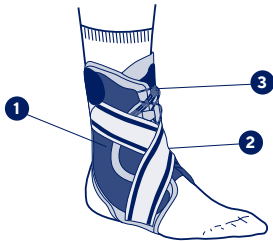
	1	19 - 22 cm	7 <sup>1/2</sup> - 8 <sup>3/4</sup> "
	2	22 - 26 cm	8 <sup>3/4</sup> - 10 <sup>1/4</sup> "
	3	26 - 30 cm	10 <sup>1/4</sup> - 11 <sup>3/4</sup> "

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	脚踝尺寸
<b>ar</b>	محيط الكاحل

<b>fr</b>	Immobilisation	Stabilisation	Proprioception
<b>en</b>	Immobilization	Stabilization	Proprioception
<b>de</b>	Ruhigstellung	Stabilisierung	Propriozeption
<b>nl</b>	Immobilisatie	Stabilisatie	Proprioceptie
<b>it</b>	Immobilizzazione	Stabilizzazione	Propriocezione
<b>es</b>	Inmovilización	Estabilización	Propiocepción
<b>pt</b>	Imobilização	Estabilização	Proprioceção
<b>da</b>	Støtte	Stabilisering	Proprioception
<b>fi</b>	Immobilisoi	Stabilisoi	Proprioseptio
<b>sv</b>	Immobilisering	Stabilisering	Proprioception
<b>el</b>	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
<b>cs</b>	Znehybnění	Stabilizace	Propriocepce
<b>pl</b>	Unieruchomienie	Stabilizacja	Propriocepcja
<b>lv</b>	Imobilizācija	Stabilizēšana	Propriocepcija
<b>lt</b>	Imobilizavimas	Stabilizavimas	Propriocepcija
<b>et</b>	Liikumatuks muutmine	Stabiliseerimine	Proprioceptsiioon
<b>sl</b>	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
<b>sk</b>	Imobilizácia	Stabilizácia	Propriocepcia
<b>hu</b>	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
<b>bg</b>	Обездвижване	Стабилизация	Проприоцепция
<b>ro</b>	Imobilizare	Stabilizare	Propriocepție
<b>ru</b>	Иммобилизация	Стабилизация	Проприоцепция
<b>hr</b>	Imobilizacija	Stabilizacija	Propriocepcija
<b>zh</b>	固定	稳定	本体感受
<b>ar</b>	التثبيت	الاستقرار	استقبال الحس العميق

fr

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## STABILISING ANKLE BRACE

### Description/Destination

This device is an ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Textile components: polyamide, polyester, neoprene, elastane.

Rigid components: stainless steel.

### Properties/Mode of action

Rigid side reinforcements to ensure ankle stability. ①

Straps that optimise the action of the reinforcements. ②

Fast, easy lace fastening. ③

Bilateral product.

### Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Ankle pain/oedema.

Sport activities resumption.

### Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

### Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device. It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral stays. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Slip the brace over the foot.

Pull the laces to close the brace, without tightening excessively to avoid any tourniquet effect. (A)

Cross the straps over the top of the foot once, then cross them a second time at the base of calf before attaching the self-fastening tabs to the sides of the ankle brace. (B-C)

In the event of discomfort in the ankle bone area, the position of the frame can be modified, by sliding it above or below the stitch located at the bottom of the frame cover.

### Care/maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Make sure to remove the lateral stays before washing.

Replace in the original location before next use. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not

tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc).

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2043601 (2023-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros